# Bruksanvisning

## KANMED<sup>®</sup> BABYBED

Art no: BB100-401F-074 SE ver 3  
2017-02-07

### Varning

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Lufttryck: 70 to 106 kPa

## 9 Service & Underhåll

NOTE: All spare parts used for replacement or repairs shall be supplied by Kanmed or its authorized distributor. The use of "non Kanmed permitted parts" will automatically reassign all safety responsibilities to the hospital. Only qualified personnel, biomed engineer or equivalent, shall attempt repairs. Do not attempt repairs with a patient in the bed.

### 9.1 Annual checks / maintenance

Check the following at least once a year:

- That cables and connectors free of defects.
- Check the Side Walls for cracks or damages.
- Check the Corners – no cracks in the plastic, screws tightened? Note, special plastic screws.
- Check the Side Wall Guides if needed for the model at hand
- Check that the Side Walls lower and raise without problems and is easily lifted away.
- Check that the Foot Pedal for Pillar height control responds without delay, that the raising and lowering function is working correctly and that it makes no unusual noise when operated and that its cable is undamaged.
- Check that the tilt function works correctly. It should not make any noises.
- Check that the wire to the Gas Spring and tilt handle is undamaged.
- Check that the wheels are tightened to the stand and not damaged in any way, that they roll without noise, and that the wheel locking function works properly.
- Check that all screws are properly tightened, especially the screws connecting the height adjusting Pillar to the wheel base, the screws on top of the Pillar, the screws holding the Bed Platform and Gas Spring.
- Check that Foot Pedal Label is not worn
- Check that the Bed Tray Labels are not worn.

## Service

NOTE

Always unplug the power before doing repair on the BabyBed.

Always make a complete function test after repairs.

### 9.2 Adjusting the release of the tilt function / Gas Spring

Hydraulic Tilt release

If the bed has mechanical tilt release then refer to version 1 of this manual for service information.

Available on [www.kanmed.se](http://www.kanmed.se).

It is a simple operation to upgrade to hydraulic tilt release – contact your distributor.

By pressing the button you activate the release pin located at the top of the gas spring inside the release head. If the tilt is not moving at all or not firmly blocked by the gas spring then you must adjust the function.

Release the gas piston from the bracket on the pillar and release the locking nut at the release head (see picture)

**Tilt is not moving:** Turn (clockwise) the gas spring one turn in insert gas piston in the bracket and push in the bolt. Test that it now moves freely and locks hard when the button is released. If not add another turn.

**The tilt is not locking:** Turn (counterclockwise) the gas spring and test.

Make a final check. Then tighten the bolts and the locking nut.

### 9.2.1 Adjusting the tilt limit.

The tilt limit can be limited by moving down the Gas Spring holder bracket on the Pillar. Make sure to tighten it properly.

A kit is available to modify the tilt range to be able to tilt the bed +/- 17°.

### 9.3 Changing or adjusting a Side Rail or Corner

See spare part list for part numbers. The Side Rail Corners are screwed from the underside with two (2) special plastic screws at each Corner. The holes in the Bed are oval allowing a tight adjustment of the Side Rails and Side Walls. It is advisable to remove the screws at min 2 Corners to fit new Corners or Side Rails.

Do not use any other screws than Kanmed part no 700-0880 because they will not have enough holding power. A screw can be screwed in and out about 6 times in the same plastic hole.

### 9.4 Adding, removing or adjusting Storage Boxes or Shelves

Loosen the screws to the Storage Box Holder. (If necessary the black decorative Plastic stripe) Move it to the new position and tighten the screws properly.

To remove or add an extra Box Holder: Raise the Pillar more than 5 cm. Press down the black plastic frame that goes around the bottom of the Pillar. Now you can remove the bolt pins or add bolt pins.

When adding Storage Boxes ensure that the Front Side Wall can be lowered sufficiently.

Cut the decorative black plastic stripes to fit. New stripes can be ordered at Kanmed.

### 9.5 Changing the Foot Pedal / electrical connections

Lay the Bed carefully on its side. Unplug the pedal from the Pillar. Unscrew the Foot Pedal and replace it. Raise the Bed and check functions and that up / down arrows are indicating correct.

#### 9.5.1 Adding an extra foot pedal or front foot pedal

Please contact Kanmed for necessary instructions and parts.

### 9.6 Changing the Pillar

Raise the Pillar electrically about 5 cm if possible.

Unplug power.

Empty the Bed and remove side walls and the Baby Tray.

Remove Storage Boxes. Press the plastic frame at the bottom of the Pillar down. Peel away the plastic strips so that you can remove the metal pins that hold accessories to the Pillar. If Pillar could not move then you will have to order new metal pins.

Remove Gas Spring from the Pillar as well as its bracket.

Loosen the bolts that hold the Bed Platform. These two bolts are lock tightened and loosening them requires quality tools and strength. Observe the earth cable.

Loosen the bolts on top of the Pillar to remove the Bed bracket. Observe the earth cable on the underside.

Unplug the height control.

Turn the Pillar/Bed upside down.

Unscrew the earth potential stud connections. (older models only)

Remove the earth protection cable under the Bed. (older models only)

Loosen screws under the wheel base and remove the foot pedal.

Unscrew the Pillar. Observe the earth cable

To assemble - redo in reverse order.

Make a full function check and an electrical safety test to ensure that the Bed Platform is properly earthed.

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| Part no  | Description                                                      | Remark                                                        |
|----------|------------------------------------------------------------------|---------------------------------------------------------------|
| BB106    | Rear mount kit standard bed                                      | Complete kit incl holder for BW3                              |
| BB107    | Rear mount kit twin beds                                         | Complete kit incl holder for BW3                              |
| BB108    | Conversion kit                                                   | Converts BB100/300 to BB101/301                               |
| BB109    | Conversion kit                                                   | Converts BB400/400F to BB401/401F                             |
| 800-0077 | Side Wall left/right BB100/BB300                                 | 29 cm height.                                                 |
| 800-0076 | Side Wall front BB100/BB300                                      | 29 cm height.                                                 |
| 800-0010 | Side Wall rear BB100/BB300                                       | 29 cm height, Grooves for tubing                              |
| 800-0101 | Side Wall left/right BB101/BB301                                 | 25 cm height.                                                 |
| 800-0100 | Side Wall front BB101/BB301                                      | 25 cm height.                                                 |
| 800-0105 | Side Wall rear BB101/BB301                                       | 25 cm height, Grooves for tubing                              |
| 800-0079 | Side Wall left/right BB400/400F                                  | 29 cm height.                                                 |
| 800-0078 | Side Wall front BB400/400F                                       | 29 cm height.                                                 |
| 800-0042 | Side Wall rear BB400/400F                                        | 29 cm height, Grooves for tubing                              |
| 800-0107 | Side Wall left/right BB401/401F                                  | 25 cm height.                                                 |
| 800-0108 | Side Wall front BB401/401F                                       | 25 cm height.                                                 |
| 800-0109 | Side Wall rear BB401/401F                                        | 25 cm height, Grooves for tubing                              |
| 800-0074 | Side Wall Guide Left side                                        | Set of 4                                                      |
| 800-0073 | Side Wall Guide Right side                                       | Set of 4                                                      |
| 800-0072 | Red side wall stopper                                            | Set of 4                                                      |
| 800-0023 | Side Rail front/rear BB100/BB300                                 | Length 40cm                                                   |
| 800-0022 | Side Rail sides BB100/BB300                                      | Length 62 cm                                                  |
| 800-0044 | Side Rail front/rear BB400/BB400F                                | Length 57,7 cm                                                |
| 800-0045 | Side Rail sides BB400/BB400F                                     | Length 72,3 cm                                                |
| 800-0011 | Corner handle for Side Rail beds                                 | With side wall guides                                         |
| 800-0110 | Corner handle BB100-400                                          | With side rails                                               |
| 800-0098 | Corner BB101-401                                                 | Without side rails                                            |
| 700-0880 | Screws for Corner Bracket                                        | Use only these special screws                                 |
| BB155    | Mattress BB100/BB300 68 cm x 46 cm                               | Plastic covered foam                                          |
| BB450    | Mattress BB400/BB400F 76 cm x 64 cm                              | Plastic covered foam                                          |
| 800-0004 | Baby Tray BB100/BB300                                            |                                                               |
| 800-0041 | Baby Tray BB400/BB400F                                           |                                                               |
| 800-0002 | Storage Box                                                      |                                                               |
| 800-0013 | Storage Box holder                                               | Includes set of bolt pin and screws                           |
| 800-0001 | Storage Box lid                                                  |                                                               |
| BB171    | Bottle holder                                                    | Not yet available                                             |
| 800-0065 | Bracket for DIN rails (fits the Side Rails)                      | Not yet available                                             |
| 800-0020 | Tent Pole for all beds                                           | Two height positions                                          |
| 800-0063 | Bracket/handle for BW3                                           |                                                               |
| 800-0502 | Special power cable for BabyWarmer                               | Length 80-100cm                                               |
| 800-0003 | Wheel top Cover                                                  |                                                               |
| BB160    | Tent for BB100/101 / BB300/301                                   |                                                               |
| BB460    | Tent for BB400/401 /400F/401F                                    |                                                               |
| 800-0501 | Foot Pedal with cable                                            |                                                               |
| 800-0029 | Bolt pin for accessories<br>(that are to be fixed on the Pillar) | Dia 7 mm, L 30 mm Screw M5.<br>Including the screws. Set of 2 |
| 800-0503 | Label Bed tray                                                   |                                                               |
| 800-0501 | Label Foot pedal                                                 |                                                               |
| BB103    | Trendelenburg kit $\pm 17$ degrees                               | New gas spring and top plate                                  |

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Kanmed AB