# Brugermanual

# KANMED<sup>®</sup> BABYBED

## CE

Varenr.: BB100-401F-073 DK ver. 3  
2017-02-14

### Vigtigt

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Storage: +10 to +40°C, non condensing.

Atmospheric pressure: 70 to 106 kPa

## 9 Technical Section - Service and Maintenance

NOTE: All spare parts used for replacement or repairs shall be supplied by Kanmed or its authorized distributor. The use of "non Kanmed permitted parts" will automatically reassign all safety responsibilities to the hospital. Only qualified personnel, biomed engineer or equivalent, shall attempt repairs. Do not attempt any service with a patient in the bed.

### 9.1 Annual checks / maintenance

Check the following at least once a year:

- That cables and connectors free of defects.
- Check the Side Walls for cracks or damages.
- Check the Corners – no cracks in the plastic, screws tightened? Note, special plastic screws.
- Check the Side Wall Guides if needed for the model at hand
- Check that the Side Walls lower and raise without problems and is easily lifted away.
- Check that the Foot Pedal for Pillar height control responds without delay, that the raising and lowering function is working correctly and that it makes no unusual noise when operated and that its cable is undamaged.
- Check that the tilt function works correctly. It should not make any noises.
- Check that the wire to the Gas Spring and tilt handle is undamaged.
- Check that the wheels are tightened to the stand and not damaged in any way, that they roll without noise, and that the wheel locking function works properly.
- Check that all screws are properly tightened, especially the screws connecting the height adjusting Pillar to the wheel base, the screws on top of the Pillar, the screws holding the Bed Platform and Gas Spring.
- Check that Foot Pedal Label is not worn
- Check that the Bed Tray Labels are not worn.

## Service

NOTE

Always unplug the power before doing repair on the BabyBed.

Always make a complete function test after repairs.

### 9.2 Adjusting the release of the tilt function / Gas Spring

Hydraulic Tilt release

If the bed has mechanical tilt release then refer to version 1 of this manual for service information.

Available on [www.kanmed.se](http://www.kanmed.se).

It is a simple operation to upgrade to hydraulic tilt release– contact your distributor.

By pressing the button you activate the release pin located at the top of the gas spring inside the release head. If the tilt is not moving at all or not firmly blocked by the gas spring then you must adjust the function. Release the gas piston from the bracket on the pillar and release the locking nut at the release head (see picture)

**Tilt is not moving:** Turn the gas spring one turn in – insert gas piston in the bracket and push in the bolt. Test that it now moves freely and locks hard when the button is released. If not add another turn

**The tilt is not locking:** Turn the gas spring one turn out and test.

Make a final check. Then tighten the bolts and the locking nut .

### 9.2.1 Adjusting the tilt limit.

The tilt limit can be limited by moving down the Gas Spring holder bracket on the Pillar. Make sure to tighten it properly.

A kit is available to modify the tilt range to be able to tilt the bed  $\pm 17$  degrees

### 9.3 Changing or adjusting a Side Rail or Corner

See spare part list for part numbers. The Side Rail Corners are screwed from the underside with two (2) special plastic screws at each Corner. The holes in the Bed are oval allowing a tight adjustment of the Side Rails and side walls. It is advisable to remove the screws at min 2 Corners to fit new Corners or Side Rails.

Do not use any other screws than Kanmed part no 700-0880 because they will not have enough holding power. A screw can be screwed in and out about 6 times in the same plastic hole.

### 9.4 Adding, removing or adjusting Storage Boxes or shelves

Loosen the screws to the Storage Box Holder. (if necessary remove the black decorative plastic stripe). Move it to the new position and tighten the screws properly.

To remove or add an extra Box Holder: Raise the Pillar more than 5 cm. Press down the black plastic frame that goes around the bottom of the Pillar. Now you can remove the bolt pins or add bolt pins.

When adding Storage Boxes ensure that the Front Side Wall can be lowered sufficiently.

Cut the decorative black plastic stripes to fit. New stripes can be ordered at Kanmed.

### 9.5 Changing the Foot Pedal / electrical connections

Lay the Bed carefully on its side. Unplug the pedal from the Pillar. Unscrew the Foot Pedal and replace it. Raise the Bed and check functions and that up / down arrows are indicating correct.

#### 9.5 1 Adding an extra foot pedal or front foot pedal.

Please contact kanmed for necessary instructions and parts.

### 9.6 Changing the Pillar

Raise the Pillar electrically about 5 cm if possible.

Unplug power.

Empty the Bed and remove side walls and the Baby Tray.

Remove Storage Boxes. Press the plastic frame at the bottom of the Pillar down. Peel away the plastic strips so that you can remove the metal pins that hold accessories to the Pillar. If Pillar could not move then you will have to order new metal pins.

Remove Gas Spring from the Pillar as well as its bracket.

Loosen the bolts that hold the Bed Platform. These two bolts are lock tightened and loosening them requires quality tools and strength. Observe the earth cable

Loosen the bolts on top of the Pillar to remove the Bed bracket. Observe the earth cable on the underside.

Unplug the height control.

Turn the Pillar/Bed upside down.

Unscrew the earth potential stud connections.(Older models only)

Remove the earth protection cable under the Bed. (Older models only)

Loosen screws under the wheel base and remove the foot pedal.

Unscrew the Pillar. Observe the earth cable

To assemble – perform instructions in reverse order.

Make a full function check and an electrical safety test to ensure that the Bed Platform is properly earthed.

## 10 Spare parts/Accessories

Part no	Description	Remark
BB106	Rear mount kit standard bed	Complete kit incl holder for BW3
BB107	Rear mount kit twin beds	Complete kit incl holder for BW3
BB108	Conversion kit	Converts BB100/300 to BB101/301
BB109	Conversion kit	Converts BB400/400F to BB401/401F
800-0077	Side Wall left/right BB100/BB300	29 cm height.
800-0076	Side Wall front BB100/BB300	29 cm height.
800-0010	Side Wall rear BB100/BB300	29 cm height, Grooves for tubing
800-0101	Side Wall left/right BB101/BB301	25 cm height.
800-0100	Side Wall front BB101/BB301	25 cm height.
800-0105	Side Wall rear BB101/BB301	25 cm height, Grooves for tubing
800-0079	Side Wall left/right BB400/400F	29 cm height.
800-0078	Side Wall front BB400/400F	29 cm height.
800-0042	Side Wall rear BB400/400F	29 cm height, Grooves for tubing
800-0107	Side Wall left/right BB401/401F	25 cm height.
800-0108	Side Wall front BB401/401F	25 cm height.
800-0109	Side Wall rear BB401/401F	25 cm height, Grooves for tubing
800-0074	Side Wall Guide Left side	Set of 4
800-0073	Side Wall Guide Right side	Set of 4
800-0072	Red side wall stopper	Set of 4
800-0023	Side Rail front/rear BB100/BB300	Length 40cm
800-0022	Side Rail sides BB100/BB300	Length 62 cm
800-0044	Side Rail front/rear BB400/BB400F	Length 57,7 cm
800-0045	Side Rail sides BB400/BB400F	Length 72,3 cm
800-0011	Corner handle for Side Rail beds	With side wall guides
800-0110	Corner handle for BB100-400 with side rail	Without side wall guides
800-0098	Corner BB101-BB401	Without side rails
700-0880	Screws for Corner Bracket	Use only these special screws
BB155	Mattress BB100/BB300 68 cm x 46 cm	Plastic covered foam
BB450	Mattress BB400/BB400F 76 cm x 64 cm	Plastic covered foam
800-0004	Baby Tray BB100/BB300	
800-0041	Baby Tray BB400/BB400F	
800-0002	Storage Box	
800-0013	Storage Box holder	Includes set of bolt pin and screws
800-0001	Storage Box lid	
BB171	Bottle holder	Not yet available
800-0065	Bracket for DIN rails (fits the Side Rails)	Not yet available
800-0020	Tent Pole for all beds	Two height positions
800-0063	Bracket/handle for BW3	
800-0502	Special power cable for BabyWarmer	Length 80-100cm
800-0003	Wheel top Cover	
BB160	Tent for BB100/101 / BB300/301	
BB460	Tent for BB400/401 /400F/401F	
800-0501	Foot Pedal with cable	
800-0029	Bolt pin for accessories (that are to be fixed on the Pillar)	Dia 7 mm, L 30 mm Screw M5. Including the screws. Set of 2
800-0503	Label Bed tray	
800-0501	Label Foot pedal	
BB103	Trendelenburg kit ±17 degrees	New gas spring and top plate

NOTE: Many other accessories and spare parts are available. Please check with your local supplier or on the Kanmed webpage [www.kanmed.se](http://www.kanmed.se) Please contact your supplier for a price list.

## 11 Warranty Statement

KANMED warrants the purchaser that the Kanmed BabyBed is free from defects in materials and workmanship for a period of **5 years** from the date of delivery. The warranty does not apply to plastic parts, mattresses, cables and tents.

The sole obligation of KANMED with respect to any such defect is limited to the repair with new or re-manufactured parts or, at the discretion of KANMED, replacement of the equipment or refunding of the purchase price.

This warranty shall not apply if the product has been modified, adjusted or repaired other than by KANMED or by organizations authorized by KANMED or modified, adjusted or repaired not in accordance with written instructions provided by KANMED. Neither shall the warranty apply if the equipment has been subject to misuse, negligence or accident.

These warranties are made on the condition that prompt notification of a defect is given to KANMED or its authorized dealers within the warranty period.

KANMED shall have the sole right to determine whether a defect exists.

KANMED shall not in any case be liable for special or consequential damages arising from the breach of warranty, breach of contract, negligence or any other legal theory.

## 12 EMC Guide-Lines

The BabyBed fulfils the applicable requirements of IEC 60601-1-2:2007 Electromagnetic compatibility (EMC).

The BabyBed shall be used in a hospital environment, but not close to HF surgical equipment or MR cameras.

**WARNING:** Use of accessories and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

**WARNING:** Portable RF communications equipment (e.g. mobile phones, radio transmitters, antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the BabyBed, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

**NOTE:** The EMISSIONS characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.

For further information about EMC compliance with the specified emissions and immunity standard, test levels, etc. please contact KANMED.

## 13 Disposal



When the Kanmed BabyBed have reached end of life, it should be recycled in accordance with the EU 2002/96/EC (WEEE) directive if applicable.

**KANMED<sup>o</sup> BABYBED**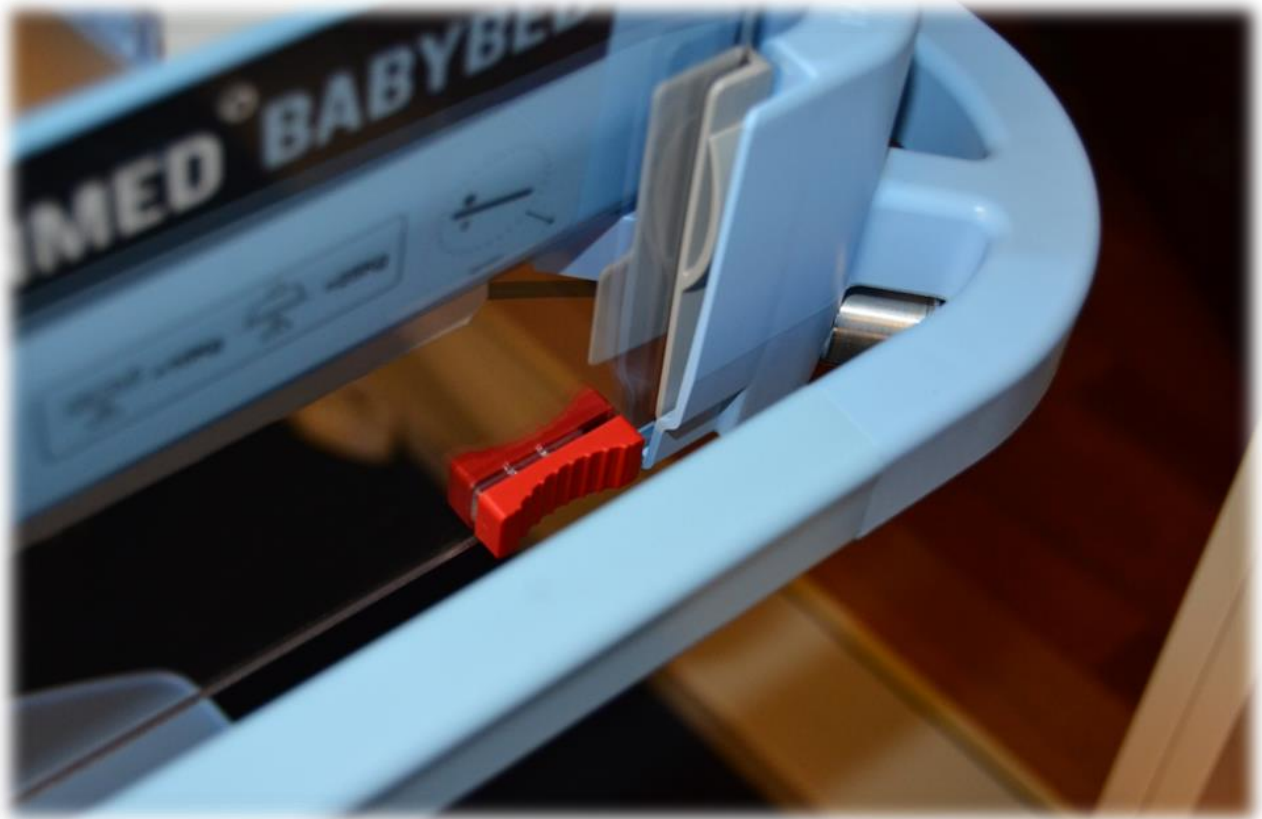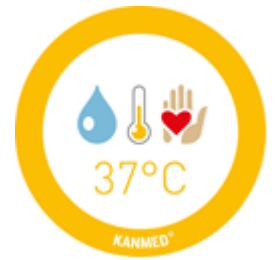

KANMED AB  
Gårdsfogdevägen 18B  
SE-168 67 BROMMA – Stockholm  
Sweden

Telephone +46 8 56 48 06 30  
Telefax +46 8 56 48 06 39

E-Mail: [info@kanmed.se](mailto:info@kanmed.se)  
Home page: [www.kanmed.se](http://www.kanmed.se)

Distributed by :