

Universal clamp. Art. no.: BW3-0837

For mounting the Kanmed BW3 control unit securely on the Kanmed beds or other beds.
 This clamp replaces the handle and is now standard on all Kanmed Baby Warmer Control Units from 2017-03-20
 Available from Kanmed at promotion price to upgrade existing units

Universal Clamp with adapter for BW3.
Art no.:BW3-0837

The clamp is mounted on the BW3 Control Unit as shown in the picture.

It can be mounted in four different positions simply by unscrewing it and place it back in the new desired position.

Universal Clamp with adapter for BW3.



Clamp mounted on the BW3, standard mount position.



Examples

Universal Clamp mounted on a Kanmed BabyBed with side rails



BW3 with Univer Clamp mounted on a Kanmed BabyBed with handle bar in front.




Universal Clamp mounted on a horizontal drip pole or similar horizontal poles, round or square, with diameter from 15 to 35 mm



Under bed Baby Warmer holder

Art. no.: BB-112
 Together with the Universal pole clamp you can mount the BabyWarmer control unit safely under the bed.
 It is wide enough to hold old BW50 BabyWarmers
 Is standard on all beds from April 2017
 BB-112 can be retrofitted on all beds.

BB-112 BabyWarmer holder mounted under the Kanmed BabyBed

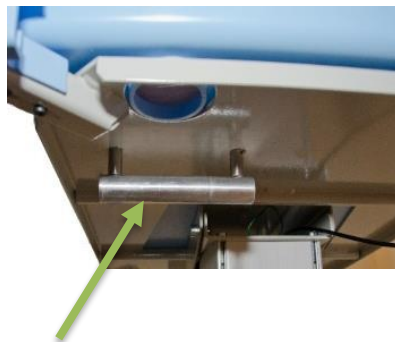

The Universal Clamp attached to BB-112



Baby Warmer is clearly visible and protected



See more next page !

<p>Bracket for 25mm Drip poles. Art. no.: 800-0527 Construction tilts the Control unit 30 degrees forward. Assures readability of the display. Requires Bracket handle or Universal clamp</p>	<p>Holds both the Universal clamp and the bracket handle</p> 	<p>800-0527 on a 25 mm pole</p> 	<p>800-0527 on a 25 mm pole with the BabyWarmer attached</p> 
<p>Rear Mount kit Art. no.: BB-106 or BB-107 for twin beds Complete kit that includes <i>one</i> drip pole. Includes 800-0527 bracket (see below) and bracket handle 800-0057 (see below) in case you do not have or don't need the Universal clamp. Fits on all Kanmed beds.</p>	<p>Universal Clamp attached to the Bracket 800-0527 that is included in the Rear Mount Kit</p> 		
<p>Bracket handle Art. no.: 800-0057 Replaces the Universal clamp for easy mounting on the Bracket for 25 mm poles (see below) or directly on Kanmed beds with side rails</p>	<p>800-0057 Bracket handle</p> 	<p>BabyWarmer with bracket handle on the Bracket for 25 mm drip poles</p> 	<p>Babywarmer with Bracket Handle on the side rails</p> 