# Brugermanual

# KANMED<sup>®</sup> BABYBED

CE

Varenr.: BB100-402-073 DK ver 7  
2023-11-17

### Vigtigt

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Storage: +10 to +40°C, non condensing.

Atmospheric pressure: 70 to 106 kPa

## 9 Technical Section - Service and Maintenance

NOTE: All spare parts used for replacement or repairs shall be supplied by Kanmed or its authorized distributor. The use of "non Kanmed permitted parts" will automatically reassign all safety responsibilities to the hospital. Only qualified personnel, biomed engineer or equivalent, shall attempt repairs. Do not attempt any service with a patient in the bed.

### 9.1 Annual checks / maintenance

Check the following at least once a year:

- That cables and connectors free of defects.
- Check the Side Walls for cracks or damages.
- Check the Corners – no cracks in the plastic, screws tightened? Note, special plastic screws.
- Check the Side Wall Guides if needed for the model at hand
- Check that the Side Walls lower and raise without problems and is easily lifted away.
- Check that the Foot Pedal for Pillar height control responds without delay, that the raising and lowering function is working correctly and that it makes no unusual noise when operated and that its cable is undamaged.
- Check that the tilt function works correctly. It should not make any noises.
- Check that the wire to the Gas Spring and tilt handle is undamaged.
- Check that the wheels are tightened to the stand and not damaged in any way, that they roll without noise, and that the wheel locking function works properly.
- Check that all screws are properly tightened, especially the screws connecting the height adjusting Pillar to the wheel base, the screws on top of the Pillar, the screws holding the Bed Platform and Gas Spring.
- Check that Foot Pedal Label is not worn
- Check that the Bed Tray Labels are not worn.

### Service

NOTE

Always unplug the power before doing repair on the BabyBed.  
Always make a complete function test after repairs.

### 9.2 Adjusting the release of the tilt function / Gas Spring

Hydraulic Tilt release

If the bed has mechanical tilt release then refer to version 1 of this manual for service information.

Available on [www.kanmed.se](http://www.kanmed.se).

It is a simple operation to upgrade to hydraulic tilt release– contact your distributor.

By pressing the button you activate the release pin located at the top of the gas spring inside the release head. If the tilt is not moving at all or not firmly blocked by the gas spring then you must adjust the function.

Release the gas piston from the bracket on the pillar and release the locking nut at the release head (see picture)

**Tilt is not moving:** Turn the gas spring one turn in – insert gas piston in the bracket and push in the bolt. Test that it now moves freely and locks hard when the button is released. If not add another turn

**The tilt is not locking:** Turn the gas spring one turn out and test.  
Make a final check. Then tighten the bolts and the locking nut .

#### 9.2.1 Adjusting the tilt limit.

The tilt limit can be limited by moving down the Gas Spring holder bracket on the Pillar. Make sure to tighten it properly. A kit is available to modify the tilt range to be able to tilt the bed  $\pm 17$  degrees

### 9.3 Changing or adjusting a Side Rail or Corner

See spare part list for part numbers. The Side Rail Corners are screwed from the underside with two (2) special plastic screws at each Corner. The holes in the Bed are oval allowing a tight adjustment of the Side Rails and side walls. It is advisable to remove the screws at min 2 Corners to fit new Corners or Side Rails.

Do not use any other screws than Kanmed part no 700-0880 because they will not have enough holding power. A screw can be screwed in and out about 6 times in the same plastic hole.

### 9.4 Adding, removing or adjusting Storage Boxes or shelves

Loosen the screws to the Storage Box Holder. (if necessary remove the black decorative plastic stripe). Move it to the new position and tighten the screws properly.

To remove or add an extra Box Holder: Raise the Pillar more than 5 cm. Press down the black plastic frame that goes around the bottom of the Pillar. Now you can remove the bolt pins or add bolt pins.

When adding Storage Boxes ensure that the Front Side Wall can be lowered sufficiently.

Cut the decorative black plastic stripes to fit. New stripes can be ordered at Kanmed.

### 9.5 Changing the Foot Pedal / electrical connections

Lay the Bed carefully on its side. Unplug the pedal from the Pillar. Unscrew the Foot Pedal and replace it. Raise the Bed and check functions and that up / down arrows are indicating correct.

#### 9.5 1 Adding an extra foot pedal or front foot pedal.

Please contact kanmed for necessary instructions and parts.

### 9.6 Changing the Pillar

Raise the Pillar electrically about 5 cm if possible.

Unplug power.

Empty the Bed and remove side walls and the Baby Tray.

Remove Storage Boxes. Press the plastic frame at the bottom of the Pillar down. Peel away the plastic strips so that you can remove the metal pins that hold accessories to the Pillar. If Pillar could not move then you will have to order new metal pins.

Remove Gas Spring from the Pillar as well as its bracket.

Loosen the bolts that hold the Bed Platform. These two bolts are lock tightened and loosening them requires quality tools and strength. Observe the earth cable

Loosen the bolts on top of the Pillar to remove the Bed bracket. Observe the earth cable on the underside.

Unplug the height control.

Turn the Pillar/Bed upside down.

Unscrew the earth potential stud connections. (Older models only)

Remove the earth protection cable under the Bed. (Older models only)

Loosen screws under the wheel base and remove the foot pedal.

Unscrew the Pillar. Observe the earth cable

To assemble – perform instructions in reverse order.

Make a full function check and an electrical safety test to ensure that the Bed Platform is properly earthed.

## 10 Spare parts/Accessories

Art. No.	Product	Additional information
BB112	Under bed BabyWarmer holder	A kit includes complete instruction. Requires a 7mm metal drill on beds before 2018. Also specify if it is for single or twin beds. BW3 may require bracket holder BW3-0837.
800-0116	Side wall <b>rear</b> BB101/301/102	Height 25cm. Beds without side rails. Groves for tubing.
800-0117	Side wall <b>rear</b> BB401/401F/402	Height 29cm. Beds without side rails. Groves tube from above.
800-0118	Side wall <b>rear</b> BB100/BB300	Height 25cm. Groves for tubing.
800-0119	Side Wall <b>rear</b> BB400/400F	Height 29cm. Groves for tubing.
800-0010	Side wall <b>rear</b> BB100/BB300.	Height 29 cm, Grooves for tubing
800-0105	Side wall <b>rear</b> BB101/BB301/102	Height 25 cm. For beds without side rails only groves for tubing
800-0042	Side Wall <b>rear</b> BB400/400F	Height 29 cm. Grooves for tubing
800-0109	Side wall <b>rear</b> BB401/401F/402	Height 25 cm For beds without side rails. Groves for tubing from side.
800-0077	Side wall <b>left/right</b> BB100/BB300	Height 29 cm
800-0101	Side wall <b>left/right</b> BB101/BB301/102	Height 25 For beds without side rails only
800-0079	Side wall <b>left/right</b> BB400/BB400F	Height 29 cm
800-0107	Side wall <b>left /right</b> BB401/401F	Height 25 For beds without side rails only
800-0076	Side wall <b>front</b> BB100/BB300	Height 29 cm
800-0100	Side wall <b>front</b> BB101/BB301	Height 25 cm For beds without side rails only
800-0078	Side wall <b>front</b> BB400/BB400F	Height 29 cm
800-0108	Side wall <b>front</b> BB401/401F	Height 25 cm For beds without side rails only
800-0072	Side wall stop red	Set of 4 pcs
800-0023	Side Rail front/rear BB100/BB300	Length 40 cm
800-0022	Side Rail sides BB100/BB300	Length 62 cm
800-0044	Side Rail front/rear BB400/BB400F	Length 57,7 cm
800-0045	Side Rail sides BB400/BB400F	Length 72,3 cm
800-0110	Corner BB100-400	Incl. 2 screws and cover under handle. For beds <b>with</b> siderails.
800-0098	Corner Handle BB101-401	Incl. 2 screws. For beds with metal rails and <b>without</b> siderails
700-0880	Screws for Corner Bracket	Use only this special screw for plastic corners
800-0004	Baby Tray BB100/BB300	Also for BB101/301/102
800-0041	Baby Tray BB400/BB400F	Also for BB401/401F/402
800-0002	Storage Box	For all bed models
800-0013	Storage Box holder	Includes set of bolt pin and screws
800-0001	Storage Box lid	For all beds
800-0003	Wheel top Cover	For all beds
800-0501	Foot Pedal	For replacement. With cable and label. Bracket not included.
800-0029	Bolt pin for pillar accessories.	Dia 7 mm, L 30 mm Screw M5. Including the screw.
700-0874	Wheel incl. bolt and bracket, with brake.	Standard on all beds
700-0884	Double wheel 95mm, lockable	Optional.
800-0048	Electrical pillar 230V	For BB100 to 401F beds only
800-0526	Electrical pillar 115 V	For BB100 to 401F beds only
800-0506	Gas spring	For beds with standard +17° tilt
BB172	Hydraulic tilt kit	Includes release hydraulic mechanism, button, bracket and dome nuts. For up grading beds with mechanical tilt.
800-0519	Hydraulic gas spring control unit	Replacement for faulty hydraulic gas spring release
800-0528	Release button (black)	Pack of 5pcs. Must be secured with Lock-Tite, glue
800-0520	Locking handle	To be used with 800-0527

## Accessories Baby Beds BB100/101/102, 300/301/302, 400/401/402, 400F/401F

Art.no	Product	Additional Information
BB155	Foam Mattress	For BB100-301. Sealed, easy to clean.
BB450	Foam Mattress	For BB400-401F. Sealed, easy to clean.
BB161	Tent BabyBed	For BB100-301. Pink/Beige.
BB160	Tent BabyBed	For BB100-301. Yellow/Terracotta.
BB162	Tent BabyBed	For BB100-301. Yellow/Blue.
BB460	Tent BabyBed twin	For B400-401F. Yellow/Terracotta.
BB170	Storage Box complete	Storage box with lid. Including necessary brackets and screws.
BB180	Metal Shelf, low complete	Metal shelf with lid. Including necessary brackets and screws. 33x33x3cm
BB190	Metal Shelf, high complete	Metal shelf with lid. Including necessary brackets and screws. 33x33x5cm
800-0518	Drip pole	Length 650mm, diameter 25mm. Including locking handle. For Kanmed Babywarmer bracket or other equipment
800-0502	Power cable for BabyWarmer	Special power cable (1m) that connects power to the Kanmed BabyWarmer, all models. Using the power outlet on the pillar under the bed tray.
BB171	Bottle Holder	Stainless steel bracket for mounting of hand disinfection bottles. Max diameter 70-80 mm.
BB173	Trendeleburg kit ( $\pm 17^\circ$ tilt)	Includes special gas spring and top plate.
BB175	Fot pedal kit Left/right	Includes bracket cable and foot pedal
BB176	Fot pedal front mount	Includes front bracket, extra cable and foot pedal

OTE: Many other accessories and spare parts are available. Please check with your local supplier or on the Kanmed webpage [www.kanmed.se](http://www.kanmed.se) Please contact your supplier for a price list.

## 1 Warranty Statement

KANMED warrants the purchaser that the Kanmed BabyBed is free from defects in materials and workmanship for a period of **5 years** from the date of delivery. The warranty does not apply to plastic parts, mattresses, cables and tents.

The sole obligation of KANMED with respect to any such defect is limited to the repair with new or re-manufactured parts or, at the discretion of KANMED, replacement of the equipment or refunding of the purchase price.

This warranty shall not apply if the product has been modified, adjusted or repaired other than by KANMED or by organizations authorized by KANMED or modified, adjusted or repaired not in accordance with written instructions provided by KANMED. Neither shall the warranty apply if the equipment has been subject to misuse, negligence or accident.

These warranties are made on the condition that prompt notification of a defect is given to KANMED or its authorized dealers within the warranty period.

KANMED shall have the sole right to determine whether a defect exists.

KANMED shall not in any case be liable for special or consequential damages arising from the breach of warranty, breach of contract, negligence or any other legal theory.

## 12 EMC Guide-Lines

The BabyBed fulfils the applicable requirements of IEC 60601-1-2:2007 Electromagnetic compatibility (EMC).

The BabyBed shall be used in a hospital environment, but not close to HF surgical equipment or MR cameras.

**WARNING:** Use of accessories and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

**WARNING:** Portable RF communications equipment (e.g. mobile phones, radio transmitters, antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the BabyBed, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.


**NOTE:** The EMISSIONS characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.

For further information about EMC compliance with the specified emissions and immunity standard, test levels, etc. please contact KANMED.

## 13 Disposal

When the Kanmed BabyBed have reached end of life, it should be recycled in accordance with the 2012/19/EU (WEEE) directive if applicable.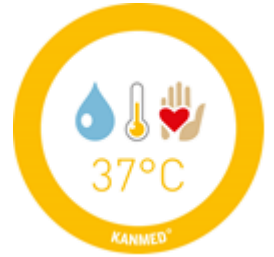

**KANMED**<sup>o</sup>  
a MedCare Visions<sup>®</sup> company

Hammarbacken 6A  
SE-191 49 Sollentuna – Stockholm  
Sweden

Telephone +46 8 56 48 06 30

E-Mail: [info@kanmed.se](mailto:info@kanmed.se)  
Home page: [www.kanmed.se](http://www.kanmed.se)

Distributed by :