Bruksanvisning

KANMED^o BABYBED

CE

Art no: BB100-080 NO vers. 7
2023-11-17

Advarsel

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8 Tekniske data

	BB100 / 101/ 102* Standard BB300 / 301 Fixed height	BB400 / 401/ 402* Twin versions BB400F / 401F Fixed height
Outer dimensions	With side rails L 85cm, W 63cm Med metallskinne B 58 cm*	Without side rails L 83cm, W 53cm
		With side rails L 95cm W 80 cm Med metallskinne B 76 cm*
		Without side rails L 93 cm W 70 cm
Bed Tray Inner dimensions	L690mm W465mm H90 mm Made of PP	L765mm W640mm H90mm Made of PP
Height from floor of Bed Tray bottom	Min 78 cm Max 108 cm. Fixed height is 80cm Ordering 9,5 cm double wheels lowers all beds about 4 cm.	
Weight	About 46 kg Fixed height about 36 kg	About 52kg Fixed height about 42 kg
Wheel base	64 cm X 87 cm	
Wheels	Diameter 125mm with brakes or 9,5 cm double wheels with brakes	
Tilt	About 17 degree. Option ± 17 degree	
Mattress	Soft foam coated	
Tent	Washable 60°C (90°C)	
Storage Box	L320mm W330mm H120mm, Lid 320x330mm	
Colors	Unisex light blue for all plastic details. Wheels and metal parts light grey	
Voltage BB100/400	230V 50Hz / 60Hz or 115V 60Hz, 150VA	
Fuses BB100/400	2xT6,3A 230V. Only replace with same type and rating	
Power outlet on Pillar BB100/101 /102 and BB400/401/402: Max 300W		
Protection BB100/101/102/400/401/402: Class I		

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Lagring: +10 to +40°C, ikke kondenserende.

Lufttrykk: 70 to 106 kPa

9 Technical Section - Service and Maintenance

NOTE: All spare parts used for replacement or repairs shall be supplied by Kanmed or its authorized distributor. The use of "non Kanmed permitted parts" will automatically reassign all safety responsibilities to the hospital. Only qualified personnel, biomed engineer or equivalent, shall attempt repairs. Do not attempt repairs with a patient in the bed.

9.1 Annual checks / maintenance

Check the following at least once a year:

- That cables and connectors free of defects.
- Check the Side Walls for cracks or damages.
- Check the Corners – no cracks in the plastic, screws tightened? Note, special plastic screws.
- Check the Side Wall Guides
- Check that the Side Walls lower and raise without problems and is easily lifted away.
- Check that the Foot Pedal for Pillar height control responds without delay, that the raising and lowering function is working correctly and that it makes no unusual noise when operated and that its cable is undamaged.
- Check that the tilt function works correctly. It should not make any noises.
- Check that the wire to the Gas Spring and tilt handle is undamaged.
- Check that the wheels are tightened to the stand and not damaged in any way, that they roll without noise, and that the wheel locking function works properly.
- Check that all screws are properly tightened, especially the screws connecting the height adjusting Pillar to the wheel base, the screws on top of the Pillar, the screws holding the Bed Platform and Gas Spring.
- Check that Foot Pedal Label is not worn
- Check that the Bed Tray Labels are not worn.

Service

NOTE

Always unplug the power before doing repair on the BabyBed.
Always make a complete function test after repairs.

9.2 Adjusting the release of the tilt function / Gas Spring

Hydraulic Tilt release

If the bed has mechanical tilt release then refer to version 1 of this manual for service information.

Available on www.kanmed.se.

It is a rather simple operation to upgrade to hydraulic tilt – contact your distributor.

By pressing the button you activate the release pin located at the top of the gas spring in the release head. If the tilt is not moving or not blocked by the gas spring then you must adjust the function.

Release the gas piston from the bracket on the pillar and release the locking nut at the release head (see picture)

Tilt is not moving: Turn (clockwise) the gas spring one turn inwards, insert gas piston in the bracket and push in the bolt. Test that it now moves freely and locks hard when the button is released.

The tilt is not locking: Turn (counterclockwise) the gas spring and test.

Make a final check. Then tighten the bolts and the locking nut.

9.2.1 Adjusting the tilt limit.

The tilt limit can be limited by moving down the Gas Spring holder bracket on the Pillar. Make sure to tighten it properly.

9.3 Changing or adjusting a Side Rail or Corner

See spare part list for part numbers. The Side Rail Corners are screwed from the underside with two (2) special plastic screws at each Corner. The holes in the Bed are oval allowing a tight adjustment of the Side Rails. It is advisable to remove the screws at min 2 Corners to fit new Corners or Side Rails.

Do not use any other screws than Kanmed part no 700-0880 because they will not have enough holding power. A screw can be screwed in and out about 6 times in the same plastic hole.

9.4 Adding, removing or adjusting Storage Boxes

Loosen the screws to the Storage Box Holder. Move it to the new position and tighten the screws properly.

To remove or add an extra Box Holder: Raise the Pillar more than 5 cm. Press down the black plastic frame that goes around the bottom of the Pillar. Now you can remove the bolt pins or add bolt pins.

When adding Storage Boxes ensure that the Front Side Wall can be lowered sufficiently.

NOTE: The aluminum pin must be inserted through the box before bolting it to the bracket.

9.5 Changing the Foot Pedal / electrical connections

Lay the Bed carefully on its side. Unplug the pedal from the Pillar. Unscrew the Foot Pedal and replace it. Raise the Bed and check functions and that up / down arrows are indicating correct.

9.6 Changing the Pillar

Raise the Pillar electrically about 5 cm if possible.

Unplug power.

Empty the Bed and remove side walls and the Baby Tray.

Remove Storage Boxes. Press the plastic frame at the bottom of the Pillar down. Peel away the plastic strips so that you can remove the metal pins that hold accessories to the Pillar. If Pillar could not move then you will have to order new metal pins.

Remove Gas Spring from the Pillar as well as its bracket.

Loosen the bolts that hold the Bed Platform. Don't forget the earth cable.

Loosen the bolts on top of the Pillar to remove the Bed bracket. Observe the earth cable on the underside.

Unplug the height control.

Turn the Pillar/Bed upside down.

Unscrew the earth potential stud connections.

Remove the earth protection cable under the Bed.

Loosen screws under the wheel base and remove the foot pedal.

Unscrew the Pillar. Observe the earth cable

To assemble - redo in reverse order.

Make a full function check and an electrical safety test to ensure that the Bed Platform is properly grounded.

10 Tillbehør / reservedeler

Art. No.	Product	Additional information
BB112	Under bed BabyWarmer holder	A kit includes complete instruction. Requires a 7mm metal drill on beds before 2018. Also specify if it is for single or twin beds. BW3 may require bracket holder BW3-0837.
800-0116	Side wall rear BB101/301/102	Height 25cm. Beds without side rails. Groves for tubing.
800-0117	Side wall rear BB401/401F/402	Height 29cm. Beds without side rails. Groves tube from above.
800-0118	Side wall rear BB100/BB300	Height 25cm. Groves for tubing.
800-0119	Side Wall rear BB400/400F	Height 29cm. Groves for tubing.
800-0010	Side wall rear BB100/BB300.	Height 29 cm, Grooves for tubing
800-0105	Side wall rear BB101/BB301/102	Height 25 cm. For beds without side rails only groves for tubing
800-0042	Side Wall rear BB400/400F	Height 29 cm. Grooves for tubing
800-0109	Side wall rear BB401/401F/402	Height 25 cm For beds without side rails. Groves for tubing from side.
800-0077	Side wall left/right BB100/BB300	Height 29 cm
800-0101	Side wall left/right BB101/BB301/102	Height 25 For beds without side rails only
800-0079	Side wall left/right BB400/BB400F	Height 29 cm
800-0107	Side wall left /right BB401/401F	Height 25 For beds without side rails only
800-0076	Side wall front BB100/BB300	Height 29 cm
800-0100	Side wall front BB101/BB301	Height 25 cm For beds without side rails only
800-0078	Side wall front BB400/BB400F	Height 29 cm
800-0108	Side wall front BB401/401F	Height 25 cm For beds without side rails only
800-0072	Side wall stop red	Set of 4 pcs
800-0023	Side Rail front/rear BB100/BB300	Length 40 cm
800-0022	Side Rail sides BB100/BB300	Length 62 cm
800-0044	Side Rail front/rear BB400/BB400F	Length 57,7 cm
800-0045	Side Rail sides BB400/BB400F	Length 72,3 cm
800-0110	Corner BB100-400	Incl. 2 screws and cover under handle. For beds with siderails.
800-0098	Corner Handle BB101-401	Incl. 2 screws. For beds with metal rails and without siderails
700-0880	Screws for Corner Bracket	Use only this special screw for plastic corners
800-0004	Baby Tray BB100/BB300	Also for BB101/301/102
800-0041	Baby Tray BB400/BB400F	Also for BB401/401F/402
800-0002	Storage Box	For all bed models
800-0013	Storage Box holder	Includes set of bolt pin and screws
800-0001	Storage Box lid	For all beds
800-0003	Wheel top Cover	For all beds
800-0501	Foot Pedal	For replacement. With cable and label. Bracket not included.
800-0029	Bolt pin for pillar accessories.	Dia 7 mm, L 30 mm Screw M5. Including the screw.
700-0874	Wheel incl. bolt and bracket, with brake.	Standard on all beds
700-0884	Double wheel 95mm, lockable	Optional.
800-0048	Electrical pillar 230V	For BB100 to 401F beds only
800-0526	Electrical pillar 115 V	For BB100 to 401F beds only
800-0506	Gas spring	For beds with standard +17° tilt
BB172	Hydraulic tilt kit	Includes release hydraulic mechanism, button, bracket and dome nuts. For up grading beds with mechanical tilt.
800-0519	Hydraulic gas spring control unit	Replacement for faulty hydraulic gas spring release
800-0528	Release button (black)	Pack of 5pcs. Must be secured with Lock-Tite, glue
800-0520	Locking handle	To be used with 800-0527

Tilbehør Baby Beds BB100/101/102, 300/301/302, 400/401/402, 400F/401F

Art.no	Product	Additional Information
BB155	Foam Mattress	For BB100-301. Sealed, easy to clean.
BB450	Foam Mattress	For BB400-401F. Sealed, easy to clean.
BB161	Tent BabyBed	For BB100-301. Pink/Beige.
BB160	Tent BabyBed	For BB100-301. Yellow/Terracotta.
BB162	Tent BabyBed	For BB100-301. Yellow/Blue.
BB460	Tent BabyBed twin	For B400-401F. Yellow/Terracotta.
BB170	Storage Box complete	Storage box with lid. Including necessary brackets and screws.
BB180	Metal Shelf, low complete	Metal shelf with lid. Including necessary brackets and screws. 33x33x3cm
BB190	Metal Shelf, high complete	Metal shelf with lid. Including necessary brackets and screws. 33x33x5cm
800-0518	Drip pole	Length 650mm, diameter 25mm. Including locking handle. For Kanmed Babywarmer bracket or other equipment
800-0502	Power cable for BabyWarmer	Special power cable (1m) that connects power to the Kanmed BabyWarmer, all models. Using the power outlet on the pillar under the bed tray.
BB171	Bottle Holder	Stainless steel bracket for mounting of hand disinfection bottles. Max diameter 70-80 mm.
BB173	Trendeleburg kit ($\pm 17^\circ$ tilt)	Includes special gas spring and top plate.
BB175	Fot pedal kit Left/right	Includes bracket cable and foot pedal
BB176	Fot pedal front mount	Includes front bracket, extra cable and foot pedal

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