


KANMED^o WARMCLOUD

PATIENT WARMING SYSTEM



Efficient patient warming system with pressure sore prevention

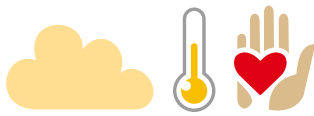

Optimal warming and pressure relief



KANMED WARMCLOUD

ensures efficient and thorough warming of the patient during long surgical procedures. The soft, disposable, single patient use mattress, reduces the risk of pressure sores – especially during very long operations.



KANMED^o WARMCLOUD

Patient Warming System with Pressure Sore Prevention

Kanmed WarmCloud is a unique system that ensures patients stay warm during surgery and reduces the risk of pressure sores. The Kanmed WarmCloud system consists of a single-use warm air mattress, a main unit and an optional remote control for operating the system. The warm air mattress that is placed under the patient, lifts the patient and creates a soft, comfortable and temperature-controlled environment.

The best solution for the patient

The system makes the cold, hard operating table warm and comfortable. This helps the patient relax. WarmCloud reduces the risk of pressure sores, especially during long procedures.

The best solution for the anaesthesia staff

Place the patient on the mattress and start the unit. Very quickly, your patient will be warmed and comfortable. The WarmCloud mattress can be used in preparation room and in post-op/ICU as well.

The best solution for the surgeon

Because the mattress is under the patient the surgeons will have easy access to surgical sites. The design prevents warming of the operating staff. The closed circuit prevents disturbance of the laminar air flow system. WarmCloud is very quiet.

The best solution for the hospital

Efficient patient warming shortens postoperative stay, since the well-known hypothermia-induced complications are eliminated. Warmcloud pressure sore prevention reduces costs and suffering.

Superior patient comfort

Is connected to the hospital's medical air outlet.

Closed circuit. No air contamination.



Under the body - disturbs nobody



Fast, simple mattress connection.



Optional Hand Controller for setting temperature and mattress pressure.

Highly efficient warming and pressure relief in one system



The quiet Control Unit can be placed at the head or foot end of the operating table. It may also be hung elsewhere using its handle and an adapter. A wheel base with tube support is available.

KANMED^o WARMCLOUD

COMPLETE SYSTEM WITH STANDARD ACCESSORIES

Art.no 9000	230VAC, 50/60Hz. (115VAC, 50/60Hz is also available).
Main Unit	
Dimensions:	Width 295 mm. Height 370 mm. Length 210 mm
Weight:	About 12 kg incl. wheelbase
Length of Air Hoses:	150 cm
Temperature range:	31°C–42°C
Pressure range:	1–10 (about 5–50 mb)
Noise level	Less than 50dB. Measured at 1m distance
Remote control (option)	
Length of cable:	2,9 m
Dimensions / weight:	Width 90 mm. Height 25 mm. Length 150 mm. Weight 200g


WARM-AIR MATTRESS, FULL LENGTH

Art no. OT-600-222	
Dimensions:	Length 200 cm. Width 55 cm
Weight:	0,95 kg
Package	10 pc. / box
Material:	EVA plastic with non-woven cover, 5 side straps

WARM-AIR MATTRESS, SHORT

Art no. OT-600-211	
Dimensions:	Length 110 cm. Width 50 cm
Weight:	0,45 kg
Package	10 pc. / box
Material:	EVA plastic with non-woven cover, 4 side straps

- ▶ The mattress is a single-use item. Other mattress sizes and accessories may be available. Invented and made in Sweden. For latest information please visit www.kanmed.se

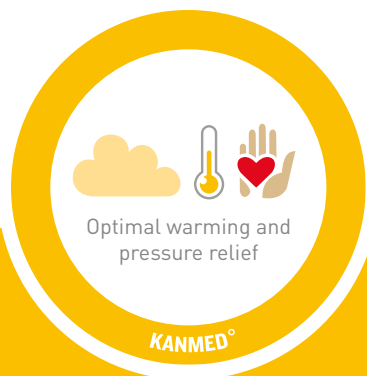

Kanmed WarmCloud has been used by more than 150 000 patients.

Visit www.kanmed.se for more information and to find your local supplier for a trial.

KANMED^o

Kanmed AB
Gårdsfogdevägen 18B, SE-168 67 Bromma, Sweden
Telephone: +46 8 564 80 630. E-mail: info@kanmed.se
www.kanmed.se