# Bruksanvisning

# KANMED<sup>®</sup> BABYBED

Art no: BB100-080 NO vers. 4  
2019-04-23

## Advarsel

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## 8 Tekniske data

	<b>BB100 / 101 Standard BB300 / 301 Fixed height</b>		<b>BB400 / 401 Twin versions BB400F / 401F Fixed height</b>	
Outer dimensions	With side rails L 85cm, W 63cm	Without side rails L 83cm, W 53cm	With side rails L 95cm W 80 cm	Without side rails L 93 cm W 70 cm
Bed Tray Inner dimensions	L690mm W465mm H90 mm Made of PP		L765mm W640mm H90mm Made of PP	
Height from floor of Bed Tray bottom	Min 78 cm Max 108 cm. Fixed height is 80cm Ordering 9,5 cm double wheels lowers all beds about 4 cm.			
Weight	About 46 kg Fixed height about 36 kg		About 52kg Fixed height about 42 kg	
Wheel base	64 cm X 87 cm			
Wheels	Diameter 125mm with brakes or 9,5 cm double wheels with brakes			
Tilt	About 17 degree. Option $\pm$ 17 degree			
Mattress	Soft foam coated			
Tent	Washable 60°C (90°C)			
Storage Box	L320mm W330mm H120mm, Lid 320x330mm			
Colors	Unisex light blue for all plastic details. Wheels and metal parts light grey			
Voltage BB100/400	230V 50Hz / 60Hz or 115V 60Hz, 150VA			
Fuses BB100/400	2xT6,3A 230V. Only replace with same type and rating			
Power outlet on Pillar BB100/101 and BB400/401:	Max 300W			
Protection BB100/101/400/401:	Class I			

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## 9 Technical Section - Service and Maintenance

NOTE: All spare parts used for replacement or repairs shall be supplied by Kanmed or its authorized distributor. The use of "non Kanmed permitted parts" will automatically reassign all safety responsibilities to the hospital. Only qualified personnel, biomed engineer or equivalent, shall attempt repairs. Do not attempt repairs with a patient in the bed.

### 9.1 Annual checks / maintenance

Check the following at least once a year:

- That cables and connectors free of defects.
- Check the Side Walls for cracks or damages.
- Check the Corners – no cracks in the plastic, screws tightened? Note, special plastic screws.
- Check the Side Wall Guides
- Check that the Side Walls lower and raise without problems and is easily lifted away.
- Check that the Foot Pedal for Pillar height control responds without delay, that the raising and lowering function is working correctly and that it makes no unusual noise when operated and that its cable is undamaged.
- Check that the tilt function works correctly. It should not make any noises.
- Check that the wire to the Gas Spring and tilt handle is undamaged.
- Check that the wheels are tightened to the stand and not damaged in any way, that they roll without noise, and that the wheel locking function works properly.
- Check that all screws are properly tightened, especially the screws connecting the height adjusting Pillar to the wheel base, the screws on top of the Pillar, the screws holding the Bed Platform and Gas Spring.
- Check that Foot Pedal Label is not worn
- Check that the Bed Tray Labels are not worn.

## Service

NOTE

Always unplug the power before doing repair on the BabyBed.

Always make a complete function test after repairs.

### 9.2 Adjusting the release of the tilt function / Gas Spring

Hydraulic Tilt release

If the bed has mechanical tilt release then refer to version 1 of this manual for service information.

Available on [www.kanmed.se](http://www.kanmed.se).

It is a rather simple operation to upgrade to hydraulic tilt – contact your distributor.

By pressing the button you activate the release pin located at the top of the gas spring in the release head. If the tilt is not moving or not blocked by the gas spring then you must adjust the function.

Release the gas piston from the bracket on the pillar and release the locking nut at the release head (see picture)

**Tilt is not moving:** Turn (clockwise) the gas spring one turn inwards, insert gas piston in the bracket and push in the bolt. Test that it now moves freely and locks hard when the button is released.

**The tilt is not locking:** Turn (counterclockwise) the gas spring and test.

Make a final check. Then tighten the bolts and the locking nut.

### 9.2.1 Adjusting the tilt limit.

The tilt limit can be limited by moving down the Gas Spring holder bracket on the Pillar. Make sure to tighten it properly.

### 9.3 Changing or adjusting a Side Rail or Corner

See spare part list for part numbers. The Side Rail Corners are screwed from the underside with two (2) special plastic screws at each Corner. The holes in the Bed are oval allowing a tight adjustment of the Side Rails. It is advisable to remove the screws at min 2 Corners to fit new Corners or Side Rails.

Do not use any other screws than Kanmed part no 700-0880 because they will not have enough holding power. A screw can be screwed in and out about 6 times in the same plastic hole.

### 9.4 Adding, removing or adjusting Storage Boxes

Loosen the screws to the Storage Box Holder. Move it to the new position and tighten the screws properly.

To remove or add an extra Box Holder: Raise the Pillar more than 5 cm. Press down the black plastic frame that goes around the bottom of the Pillar. Now you can remove the bolt pins or add bolt pins.

When adding Storage Boxes ensure that the Front Side Wall can be lowered sufficiently.

NOTE: The aluminum pin must be inserted through the box before bolting it to the bracket.

### 9.5 Changing the Foot Pedal / electrical connections

Lay the Bed carefully on its side. Unplug the pedal from the Pillar. Unscrew the Foot Pedal and replace it. Raise the Bed and check functions and that up / down arrows are indicating correct.

### 9.6 Changing the Pillar

Raise the Pillar electrically about 5 cm if possible.

Unplug power.

Empty the Bed and remove side walls and the Baby Tray.

Remove Storage Boxes. Press the plastic frame at the bottom of the Pillar down. Peel away the plastic strips so that you can remove the metal pins that hold accessories to the Pillar. If Pillar could not move then you will have to order new metal pins.

Remove Gas Spring from the Pillar as well as its bracket.

Loosen the bolts that hold the Bed Platform. Don't forget the earth cable.

Loosen the bolts on top of the Pillar to remove the Bed bracket. Observe the earth cable on the underside.

Unplug the height control.

Turn the Pillar/Bed upside down.

Unscrew the earth potential stud connections.

Remove the earth protection cable under the Bed.

Loosen screws under the wheel base and remove the foot pedal.

Unscrew the Pillar. Observe the earth cable

To assemble - redo in reverse order.

Make a full function check and an electrical safety test to ensure that the Bed Platform is properly grounded.

## 10 Tillbehør / reservedeler

Part no	Description	Remark
BB-112	Kit for mounting the BW3 under the bed	Standard on newer beds.
800-0077	Side Wall left/right BB100/BB300	29 cm height.
800-0076	Side Wall front BB100/BB300	29 cm height.
800-0010	Side Wall rear BB100/BB300	29 cm height, Grooves for tubing
800-0101	Side Wall left/right BB101/BB301	25 cm height.
800-0100	Side Wall front BB101/BB301	25 cm height.
800-0105	Side Wall rear BB101/BB301	25 cm height, Grooves for tubing
800-0079	Side Wall left/right BB400/400F	29 cm height.
800-0078	Side Wall front BB400/400F	29 cm height.
800-0042	Side Wall rear BB400/400F	29 cm height, Grooves for tubing
800-0107	Side Wall left/right BB401/401F	25 cm height.
800-0108	Side Wall front BB401/401F	25 cm height.
800-0109	Side Wall rear BB401/401F	25 cm height, Grooves for tubing
800-0072	Red side wall stopper	Set of 4
800-0023	Side Rail front/rear BB100/BB300	Length 40cm
800-0022	Side Rail sides BB100/BB300	Length 62 cm
800-0044	Side Rail front/rear BB400/BB400F	Length 57,7 cm
800-0045	Side Rail sides BB400/BB400F	Length 72,3 cm
800-0110	Corner handle BB100-400 (May 2017)	With side rails. No guides
800-0098	Corner BB101-401	Without side rails
700-0880	Screws for Corner Bracket	Use only these special screws
BB155	Mattress BB100/BB300 68 cm x 46 cm	Plastic covered foam
BB450	Mattress BB400/BB400F 76 cm x 64 cm	Plastic covered foam
800-0004	Baby Tray BB100/BB300	
800-0041	Baby Tray BB400/BB400F	
800-0002	Storage Box	
800-0013	Storage Box holder	Includes set of bolt pin and screws
800-0001	Storage Box lid	
BB171	Bottle holder	
800-0065	Bracket for DIN rails (fits the Side Rails)	
800-0020	Tent Pole for all beds	Two height positions
800-0057	Bracket/handle for BW3	
800-0502	Special power cable for BabyWarmer	Length 80-100cm
800-0003	Wheel top Cover	
BB160	Tent for BB100/101 / BB300/301	
BB460	Tent for BB400/401 /400F/401F	
800-0501	Foot Pedal with cable	
800-0029	Bolt pin for accessories (that are to be fixed on the Pillar)	Dia 7 mm, L 30 mm Screw M5. Including the screws. Set of 2
800-0503	Label Bed tray	
800-0501	Label Foot pedal	
BB103	Trendelenburg kit ±17 degrees	New gas spring and top plate

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Kanmed AB